

Using the Ling Sound Test with ADHEAR

The Ling sound test is used to check that an individual has access to sounds across the full speech spectrum.

Detection: The individual indicates a sound is heard

Identification: The individual repeats the sound or points to a card

Directions

1. Individuals using ADHEAR for unilateral hearing loss, block, (as best possible) normal hearing ear
2. Say each sound, one at a time, and in no particular order. Use a normal speaking volume. Remove all lipreading and visual cues.
3. Check distance hearing by delivering the sounds from 2-3 meters away.
4. Adjust volume if required.
5. Document the results.



Keep track of responses

Date	ah	m	oo	ee	sh	ss

- ✓ = Correct response
- = No response
- () = If the wrong sound is repeated back, the person administering the test should write the sound produced. (e.g. if the sound presented is "sh" and the individual using ADHEAR says "s", write "s" in the box.

MED⁹EL

ah



MED⁹EL

m



MED⁹EL

oo



MED⁹EL

ee



MED⁹EL

sh



MED⁹EL

ss

